NORMEN VAN REFERENTIE OP PBM
Nr. 89/686/EEC

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STANDARDS OF REFERENCE FOR PPE no 89/686/EEC

Category I: Minor risks. PPE of simple design for protection against minor risks which have no affects on the user safety.

Category II: Intermediary risks.

Category III: Major risks. PPE of complex design intended to protect against serious and irreversible health risks or mortal risks. Classified in different types:

Type 3 EN 14605+A1(2009) tight to chemical liquids

Type 4 EN 14605+A1(2009) tight to chemical spray

Type 5 EN 13982-1:2005 tight to airborne solid particulates

Type 6 EN 13034+A1(2009) limited splash protection

Coverall with elasticated hood:

- With self-adhesive zip-fastener
- Double Adhesive flap at the fastening
- Elastic at the waist, ankles and wrists
- Individual pouch

Field of application:

- Tight to chemical liquids
- Tight to airborne solid particles

A piece of protective clothing, single-use only, designed to protect the users against hazardous substance or potential contamination. The product protects against chemical liquids and airborne solid particles. Full body chemical protection can be achieved only if this coverall is in combination with other PPE such as respiratory equipment, safety boots and gloves. Waist and ankles need to be taped by adhesive (consult the manufacturer).

Conditions of use:

- Always zip up entirely
- Cover the boots and the gloves with the sleeves and the legs of the coverall.
- Adjust the sleeves and the legs by the elastic closures.

It is recommended to cover by an adhesive tape all the openings at the face, sleeves and ankles.

- Put the first flap of the coverall on the second one by adjusting the double-sided adhesive gently, and then press the flap tightly. Move the double flap left and right to ensure that it is perfectly watertight. Attention: The protective paper of the double-sided adhesive has to be taken off before you fix the flap on the coverall.

Limitations of use:

Exposure to certain very fine particles, intensive liquid sprays and splashes of hazardous substance may require coveralls of higher protection than that offered by Polyvet Extra 100PE04. Please ensure that you have chosen the garment suitable for your job and check if the chemicals you handle have been tested on the coverall (the use is limited to the chemical product list enclosed). This garment is not suitable for protection against infectious agent contamination. Prolonged wearing of the coverall may cause thermic stress, please limit the duration of use.

For further information, please contact your supplier of the garment or Texus Medical Bulgaria. The user shall be the sole judge for the correct combination of Polyvet Extra 100PE04 and ancillary equipment (gloves, boots, respiratory protective equipment etc).

Storage and Disposal:

Single-use piece of clothing. Discard after using.

- It should be stored in its original packaging, sheltered from direct sunlight and damp, at ambient temperature.
- Avoid storing this product at a temperature of -10° and +40°.
- Expiry date up to 5 years after the date of production.
- Disposal: The polyethylene composing this garment and the product packaging present no danger for the environment and can be incinerated. Restrictions (according to the regulations in force) on the disposal are applied solely in case of contamination during use.

The manufacturer shall not accept any responsibility for any improper use or disposal of garments.

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Performance chart of Polyvet Extra 100PE04	
Chemical protection type 3 according to EN 14605+A1(2009)	Chemical protection type 5 according to EN EN 13982-1:2005
TECHNICAL FEATURES Properties	
Abrasion resistance	
Flex cracking resistance	
Trapezoidal tear resistance MD/XD	
Perforation strength	
Tensile strength	
Seams resistance	
Chemical permeation	
Sulphuric acid 30% (H2SO4 30%)	
Sodium hydroxide 10% (NaOH 10%)	
Butane -1-ol	
Liquid penetration resistance (spray test type 3)	

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Airborne solid particles (test Type 5)

*Attention: with complementary equipments: facing mask P3, gloves, boots fixed with adhesive tape, double flap correctly adjusted.

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